DEUTSCH

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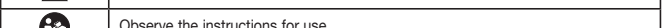
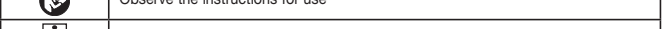
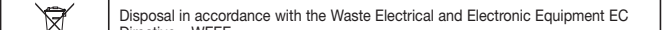
ENGLISH

Intended use

The basal thermometer is a special digital thermometer designed to measure the basal temperature in women of childbearing age. The maximum temperature measured with the sensor is saved until the next measurement. Transferring the measurements to the Ovy app enables fertile and non-fertile days in the cycle to be predicted, as well as periods.

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

	Attention
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
IP 27	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water
	Batch designation
	Permissible storage and transport temperature and humidity

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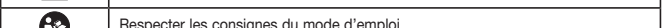
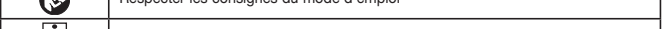
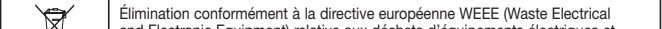
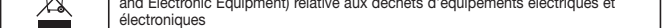
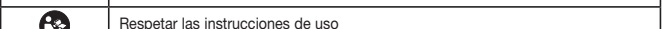
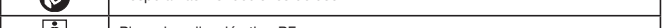
Electromagnetic Compatibility Information

The device satisfies the EMC requirements of the international standard IEC 60601-1-2. The requirements are satisfied under the conditions described in the table below. The device is an electrical medical product and is subject to special precautionary measures with regard to EMC which must be published in the instructions for use. Portable and mobile HF communications equipment can affect the device. Use of the unit in conjunction with non-approved accessories can affect the device negatively and alter the electromagnetic compatibility. The device should not be used directly adjacent to or between other electrical equipment.

Table 1

Guidance and declaration of manufacturer-electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
<i>Emissions test</i>	<i>Compliance</i>	<i>Electromagnetic environment – guidance</i>
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	N/A	

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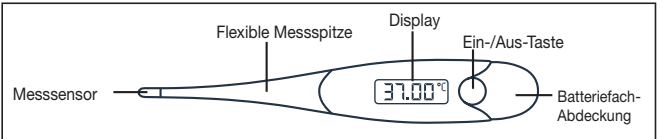
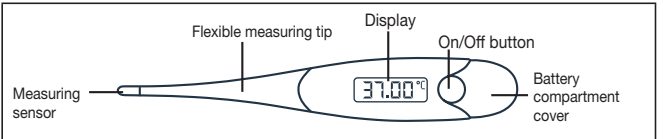
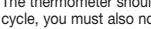
- This device meets the requirements of the EU Directive 93/42/EEC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act), the European Standard EN 12470-3: Clinical thermometers – Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN60601-1-2, and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this device. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

The basal thermometer

Measuring your basal temperature

Take your temperature at the same time, as soon as you wake up and before you get out of bed. You should have slept for at least 4 hours before the measurement. If you have slept for less than 4 hours before the measurement, enter the amount of sleep you have had in the Ovy app or activate »Disrupted sleep«. Do not eat anything before the measurement and avoid physical exertion. You can make oral, rectal or vaginal measurements, but you should always take the measurements in the same place in a cycle. We recommend taking the measurement under your tongue with your mouth closed. The thermometer should not be changed during the cycle. If you change the thermometer during the cycle, you must also note it as a disruption, just like a change to the place of measurement. For this cycle, a reliable evaluation is then no longer guaranteed.

	Press the On/Off button to switch on the device. A short acoustic signal will announce that the thermometer is switched on. The thermometer will carry out a self-test for about 2 seconds. During the selftest all segments of the display will appear.
--	--

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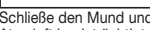
	Then the reference reading 36,5 °C appears. Then the measurement symbol “°C” flashes and “Lo” is shown on the display. The thermometer is now ready for use. Insert the tip of the sensor into one of the two heat pockets under your tongue, to the left or right of the root of your tongue. The measurement sensor must have good contact with tissue.
	

Close your mouth and breath calmly through your nose so that the air from your breathing does not influence the measurement. During the measurement, the current temperature is displayed continuously, and the “°C” symbol flashes on and off. The measurement is completed when a stable temperature has been reached. When this stage has been reached, an acoustic signal is heard, the “°C” symbol stops flashing and the temperature measured is displayed. We recommend a measuring time of 3 minutes. If the temperature stability is reached earlier, the acoustic signal sounds. Please always wait until the end of the measurement before you remove the thermometer from the place of measurement. The measured value is saved until the next measurement. The device switches off automatically approx. 10 minutes after the end of the measurement. You can also switch off the thermometer before this time by pressing the On/Off button.

Memory

The last measured value is automatically saved. To display this value, press the On/Off button for more than 2 seconds. The last measured value is then shown. Once the button is released, the reference value of 36,5°C and “Lo” are shown on the display. The thermometer is ready to take a measurement and the previously stored measured value is automatically replaced by the new measured value.

Replacing the battery

- Replace the battery when the battery status indicator appears on the display:
- Slide back the battery compartment cover to remove it.
- Pull the battery holder carefully out of the housing by approx. 1 cm.
- Use a non-metallic object to push the battery out of its holder.
- Insert a new battery (type LR41 1.5V) with the + symbol facing upwards.
- Push the battery holder back into the housing and reattach the battery compartment cover. Ensure that you do not damage or displace the seal.

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Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries. The codes below are printed on batteries containing harmful substances: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. This battery contains: Pb, Hg.

⚠️Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- ⚠️ Choking hazard! Small children may swallow and choke on batteries or the battery compartment. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- ⚠️ Risk of explosion! Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

Cleaning the device

Clean the basal thermometer before and after each use with a soft cloth and isopropyl alcohol diluted with water or cold soapy water. The device can be immersed in water or a disinfectant solution to clean it. The maximum immersion time is 30 minutes. Do not sterilise the device by boiling, with gas or with an autoclave.

Storage and disposal

Please store the thermometer in the original protective cover when it is not in use. The accuracy of this thermometer has been carefully checked and it has been developed with a long service life in mind. If the device is used for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

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The Ovy app is free for iOS and Android™

The Ovy app enables you to record your entire cycle and keep an eye on it at any time on your smart-phone. Ovy is designed for women who want to have children. The app is available to download for free in the Apple App Store and Google Play. The Ovy app is a certified Class 1 medical device.

How does Ovy work?

The Ovy app works out your cycle automatically, without pen and paper. This includes your period, ovulation and your fertile and non-fertile windows. To do this, take your temperature when you wake up in the morning and enter this value (to two decimal places) in the Ovy app. To make the prediction even more accurate, add other body signals each day. You should ideally start recording this on the first day of your period; i.e. the first day of your menstrual cycle. However, you can also start recording on any day in your cycle. It is important that you record when potential external temperature influences such as a fever, consumption of alcohol or a lack of sleep have occurred.

You can also enter your basal temperature, body signals and influencing factors at a later time, but the temperature measurement should have been taken on the relevant day.

The Ovy app is based on a self-learning algorithm. This means that the more often you enter your basal temperature, the more accurately Ovy will be able to predict your fertile days and your next period.

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Website: www.ovyapp.com
Contact: hello@ovyapp.com
Facebook: Ovy
Instagram: @ovyapp

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Do you have any questions about the Ovy app?

The Ovy team is on hand to help in such cases and you are welcome to e-mail us at hello@ovyapp.com. The Ovy app has a contact function in your profile, which you can also use to send us your questions and feedback directly. The Ovy website and the information sections in the app already offer some answers and diverse background information about your cycle.

Website: www.ovyapp.com
Contact: hello@ovyapp.com
Facebook: Ovy
Instagram: @ovyapp
Twitter: @ovyapp

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Table 4

Recommended separation distances between portable and mobile RF communications equipment and the device		
The device is intended for use in an electromagnetic environment in which radiated therefore disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.		
Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)	
	800 MHz to 800 MHz	800 MHz to 2,7 GHz
	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0,01	0,12	0,23
0,1	0,38	0,73
1	1,2	2,3
10	3,8	7,3
100	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.		
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.		
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.		

791_38_OT20-2018-05-30_04_IM1a_BEU

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Beurer GmbH